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Art.Nr. 208 1MS1

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SYNERGY FOR ABILITY

### General

**Call Easy** is a mobile call pager based on the Easywave system from ELDAT. It can learn up to 32 transmitter signals. When a stored signal is received, the display illuminates to show a predefined text which has been set with PC software. Pressing briefly the blue button on the front displays all received calls. A longer press confirms the current displayed call, deletes it from the list and sends out a different Easywave signal.

### Technical Specs

Power Sup.:	5V DC (Micro – USB)
Channels:	32
Size:	105 x 65 x 23mm
Weight:	approx. 200g
Battery:	Li-Po, 3,7V, 1600mAh

### Scope of delivery

Call Easy Receiver
Micro-USB Cable
USB Power Adapter
User Manual
Wall Mounting

### Store / Delete transmitters

- Using the paper clip, press the programming button located within a small hole on the back of the pager (see picture...). The display will then show „Prog Mode“. Press the blue button to change to the first memory location. The first line shows the location number and whether it is “free” or “used” when there is an already stored signal.

In the second line you can see the programmed text.

- Now press the blue button again for more than 1,5s. The display changes to „x Learn“ (x is the chosen number) and Call Easy is now waiting for an Easywave signal.

### Learn

- Press the button on the transmitter that you would like to store.

**(NOTE: if there is already a stored signal it will be overwritten. If the same signal is there the display shows “is used”!)**

- Call Easy will restart now and work with the new signal.

- If you want to store more signals, repeat the procedure.

### Delete

- Press the blue button again and hold it depressed until „DEL“ is shown in the second line.

- Call Easy restarts and the memory location is free now.

### Normal Use

#### Receiving Calls

When Call Easy receives a stored signal, it will display the text and start vibrating and beeping three times. This is repeated twice after 10s. The first line shows “Calls” and the number of overall received calls.

Use the blue button with a brief press to toggle between the received calls.

#### Confirm Calls

Press and hold the blue button to confirm the current displayed call. This will delete that call from the list and will send a signal which can be used to program another Easywave receiver (for example RCP04).

#### ON/OFF / Mute

Call Easy can be turn off or set to a “Mute” where no signal is received anymore. To do this please press and hold the power button for 3s or the blue button for approx. 10s until „Mute“ is displayed. Do the same again to revert back to normal use mode.


**NOTE: the Mute function is only available when all received calls are confirmed and the display is dark!**

#### Battery

You can check the current battery percentage status by pressing the blue button briefly after all received calls are confirmed (display dark) or in Mute-Mode.

While charging you can additionally see a flash if the blue button is pressed briefly. Low battery is indicated with “Bat.” in the display and a repeating tone.

### Program Texts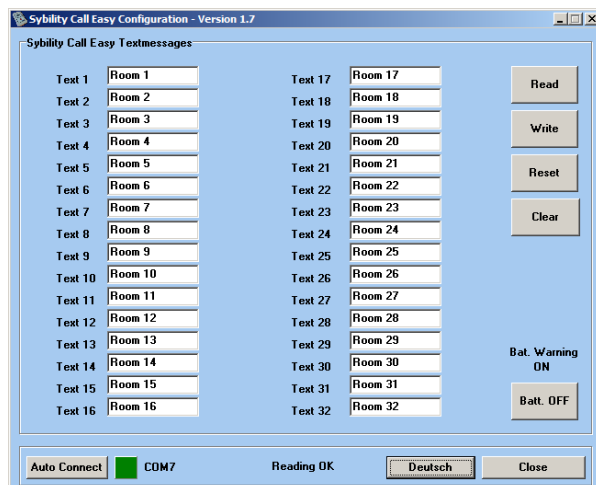

Connect the Call Easy with your computer and run the program. Click on “Auto Connect” and the program will search for the Call is Easy. The COM port will be shown if it was found.

**Read** Reads all texts from Call Easy and display them.

**Write** Writes all displayed texts into Call Easy. Empty fields will be set to „Room x“ automatically.

**Reset** Deletes all texts in Call Easy and changes to default „Room x“. It will also read and display the texts afterwards.

**Clear** Clears all fields in the program (not in Call Easy).

**Bat. OFF** This will turn off the low battery indication until the next restart of the Call Easy, for example when you would like to send the device.

**Deutsch/English** Changes the displayed language (if a Call Easy is connected it will also change language).

**Close** Closes the program.

You can download the program at [www.sybility.de](http://www.sybility.de).